


Kendall™ Neonatal-Pediatric Prewired Monitoring Electrodes

Maximize your results

1 Prep the skin

- Prep skin, as per facility protocol.
- Apply electrode to clean, dry site.

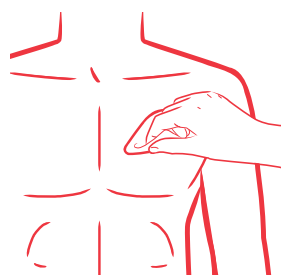

2 Application

- Do not open package until immediate usage.
- Apply the electrode to the skin.
- Ensure full surface of electrode is in contact with the skin.
- Apply gentle pressure to ensure proper adhesion.
- Plug connectors to cable.



3 Tracing

- Make sure lead wire connection is clean and secure.
- Change as per facility protocol and ensure it is less than 72 hours.



4 Removal and Disposal

- Peel electrode gently lifting from the edge.
- Intended for single use.
- **DO NOT** pull on the wire to remove electrode.*

***Caution:** This step is extremely crucial in ensuring proper removal of electrodes and must be followed cautiously.

